Cavitron

**DENTSPLY® CAVITRON® JET AIR POLISHING INSERTS**

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DENTSPLY

## DIRECTIONS FOR USE

The Cavitron JET Air Polishing Insert is for use on Cavitron PROPHY JET® (Type Gen. 122) Air Polishing Units and 30 kHz Cavitron® JET Combination Systems for the removal of extrinsic stain, dental plaque and soft debris while polishing the tooth surfaces.

### INDICATIONS FOR USE

- Removal of extrinsic stains: e.g. tobacco, coffee and tea, chlorhexidine.
- Prophylaxis of orthodontic patients.
- Preparation of tooth surfaces prior to bonding and sealant procedures.

### CONTRAINDICATIONS

- 2 • Do not direct the air-polishing stream at soft tissue or into the

sulcus. Tissue emphysema has been reportedly caused when the air/water/powder stream was directed at the soft tissue or into the sulcus.

### WARNING

- Use only with DENTSPLY Cavitron PROPHY JET Air Polishing units (Type Gen. 122) and 30 kHz Cavitron JET Combination Systems.
- Use only with DENTSPLY Cavitron Prophy Powders.
- Handle insert with care. Improper handling of insert, specifically the insert tip, may result in injury and/or cross contamination.
- It is strongly recommended that this insert be used in accordance with the current CDC Guidelines for Infection Control in dental healthcare settings.
- Failure to follow properly validated sterilization processes and approved aseptic techniques for Cavitron JET Air Polishing Inserts may result in cross contamination.

### **PRECAUTIONS**

- Never operate the system without fluid flowing through the handpiece.
- A dampened 2x2-gauze pad can be placed on the patient's lip to protect the skin and increase patient comfort.
- Direct contact of prophy powder with surfaces and marginal areas of dental restorations should be avoided.

**ADVERSE REACTIONS:** None Known.

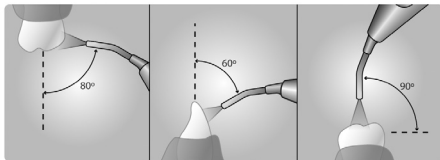
### **DIRECTIONS FOR USE**

- Place a 2x2 gauze pad on lip and position saliva ejector.
- Place nozzle tip 3 to 4 mm slightly apically on the incisal to middle third of the tooth, using constant circular motion, interproximal to interproximal

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- Polish 1 to 2 teeth per 1 to 2 seconds and rinse.
- Use your hand and patient's cheeks or lips to contain aerosols.
- Rinse slurry from patient's mouth as needed.

## INFECTION CONTROL PROCEDURES

### WARNINGS

- All devices should be cleaned and sterilized before each use. For all the countries that require a three-step process, in which disinfection is required prior to sterilization, all devices must be cleaned, disinfected and sterilized before use.
- High level disinfection by itself is not appropriate for these devices.
- Do not allow device to exceed 279° F/137° C.

### LIMITATIONS ON PROCESSING

- Repeated processing has minimal effect; end of life is normally determined by wear and damage due to use.
- Always use a pH neutral cleaning solution (such as Sultan Healthcare ReSURGE® which is recommended) / disinfecting detergent or solution. Product can be damaged by alkaline and

acidic detergents.

- Failure to follow the cleaning solution recommended concentration rates and contact times may result in rapid deterioration of the material.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.

### INSTRUCTIONS

#### **Point of use:**

- JET Air Polishing inserts must be removed from the handpiece before processing.
- Do not allow residue or any form of contamination to dry on device.
- Following use, device should be reprocessed as soon as is reasonably practical.

### **Cleaning and Disinfection: Automated – Instrument Washer-Disinfector**

- If soil is heavy or difficult to remove, manual pre-cleaning is recommended (See “Cleaning: Manual” section.)
- Remove excess powder from the air polishing insert nozzle by inserting the applicable cleaning wire(s) provided with your system into the center tube and moving the wire back and forth several times. Insertion may require the aid of pliers or tweezers in order to avoid kinking or bending the wire. Remove wire for cleaning and sterilization.
- Follow manufacturer instructions for instrument washer solution, observing concentration rates and contact times as well as the instrument washers recommended cleaning and/or thermal high level disinfection cycle (1 minute at 90°C). (Validated parameters can be found at [dentsply.com](http://dentsply.com))
- The use of an instrument cassette during an automated

cleaning/disinfection is recommended to reduce the risk of instrument damage.

- Place the device into the instrument washer and begin the washer disinfector cycle.
- After the washer cycle ends remove the instruments from the instrument washer.
- If a drying cycle is not used, dry the insert using a dry, lint-free disposable cloth or disposable wipe. An insert left standing wet may stain, corrode, or clog.
- Pay special attention to rinse all holes / cannulas. Cannulas may be rinsed several times using a disposable syringe.

### **Cleaning: Manual**

- Remove excess powder from the air polishing insert nozzle by inserting the applicable cleaning wire(s) provided with your system into the center tube and moving the wire back and forth several times. Insertion may require the aid of pliers or tweezers

in order to avoid kinking or bending the wire. Remove wire for cleaning and sterilization.

- Remove contamination by scrubbing the instrument manually with a surfactant or detergent and water observing manufacturer instructions for concentration rates and contact times paying attention to holes/cannulas.
- Rinse the insert under tepid (potable) running water for 30 seconds to remove cleaning agent: a disposable syringe may be used to rinse the holes/cannulas.
- Dry the insert using a dental instrument dryer, a dry, lint-free disposable cloth or disposable wipe. An insert left standing wet may stain, corrode, or clog.
- Check that all debris has been removed from the insert. If not, repeat the cleaning process.

#### **Disinfection: Manual**

6

- After cleaning thoroughly, wipe all insert surfaces with a

disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution observing manufacturer instructions for concentration rates and contact times.

- Pay special attention to disinfect all holes/cannulas.
- Rinse the insert under tepid (potable) running water for 30 seconds to remove disinfectant solution; a disposable syringe may be used to rinse the holes/cannulas.
- Dry the insert using a dental instrument dryer, a dry, lint-free disposable cloth or disposable wipe. An insert left standing wet may stain, corrode, or clog.

#### **Maintenance, inspection and testing:**

- Visually inspect to ensure all contamination is removed.
- Visually inspect all o-rings for wear. Replace worn o-rings (Refer to O-Ring Replacement Kit Instructions). Lubricate o-rings with water before use.

- Check for distortion, damage and wear. Discard damaged, worn or corroded instruments.

**Packaging:**

Place instrument or instrument cassette in a paper or paper/plastic steam sterilization pouch or a sterilization wrap (which can be either woven or unwoven). Instruments may be loaded into dedicated instrument trays.

**Sterilization:**

Use a steam autoclave. Place bagged instruments into the steam autoclave, according to manufacturer's recommendations.

Cassette Pressure Pulse Steam Sterilization

Full Cycle: Bagged -135°C (275°F) for 3.5 minutes

Pre-vacuum Steam Sterilization

Full Cycle: Bagged -132°C (270°F) for 3 minutes

Alternate Method:

Place non-bagged instruments into the steam autoclave and run at the listed cycles.

NOTE: Instruments sterilized unbagged should be used immediately.

**Drying:**

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes.

**Storage:**

To maintain sterility, instruments should remain bagged until ready for use.

**WARRANTY**

Cavitron® JET Air Polishing Inserts manufactured by DENTSPLY Professional are warranted against defects arising from faulty materials or workmanship for ninety days from the date of purchase. This warranty extends to inserts purchased from an authorized DENTSPLY distributor and only to the original purchaser and is valid only for inserts purchased in the United States (U.S.) and for those countries without specific warranty requirements. For inserts purchased outside the U.S. in countries with specific warranty requirements, those applicable country warranties will apply. This warranty is subject to the following conditions:

- 1) Inserts must not be subjected to abuse or improper use or application.
- 2) Inserts must not be subjected to abuse or improper cleaning, disinfection, or sterilization procedures.
- 3) Bending or reshaping the inserts will void the warranty.

4) Using inserts in units other than CAVITRON air-polishing systems will void the warranty.

There are no warranties, expressed or implied beyond the description on the face hereof. DENTSPLY neither assumes, nor authorizes any person to assume for it, any other liability for incidental and consequential damages resulting from the use of the insert.

To receive reimbursement for any defective Cavitron JET Air-Polishing insert, you must send the defective product within the warranty period to the dealer or manufacturer with specification of the defect and receipt of purchase.

**FOR TECHNICAL SUPPORT:**

**Call DENTSPLY PROFESSIONAL PRODUCT SERVICES  
DEPARTMENT at 1-800-989-8826.**

**LEARN MORE ABOUT CAVITRON JET AIR-POLISHING  
INSERTS**

For more information about Cavitron Jet Air-Polishing Inserts,  
please visit [www.densply.com](http://www.densply.com).

**DISPOSAL:**

Dispose of inserts in accordance with state and local laws.

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DENTSPLY DeTrey GmbH