# Cavitron® Brand Prophy Powder

## DIRECTIONS FOR USE

For Dental use only

ENGLISH

**Caution:** United States Federal law restricts this device for sale by, or on the order of, a licensed dental professional.

**DESCRIPTION:** Prophy-Jet® Prophy Powder is a flavored sodium bicarbonate powder. JET-Fresh® Prophy Powder is a flavored aluminum trihydroxide powder. Both prophy powders are for professional air polishing prophylaxis procedures.

**NOT FOR INTERNAL USE.**

**INDICATIONS AND USAGE:** For use in conjunction with an air polishing system, for removal of a variety of extrinsic stains, e.g. tobacco, coffee and tea, chlorhexidine. Prophylaxis of orthodontic patients. Preparing tooth surfaces prior to bonding and sealant procedures. JET-Fresh powder is recommended for patients on a sodium restricted diet.

**CONTRAINDICATIONS:** Patients who have severe respiratory illness should consult their physician before undergoing air polishing prophylaxis procedures. Use is not recommended for patients with known allergies to the components of Cavitron Brand Prophy Powders.

Manufactured by  
DENTSPLY Professional  
DENTSPLY International

1301 Smile Way  
York, PA 17404 USA  
Customer Service: 1.800.989.8826

Imported and Distributed in  
Canada by  
DENTSPLY Canada  
Woodbridge, Ontario  
L4L 4A3

DENTSPLY DeTrey GmbH  
De-Trey-Str. 1  
78467, Konstanz  
Germany

**WARNINGS:** Do not direct the air polishing stream at soft tissue or into the sulcus. Tissue emphysema has been reportedly caused when the air/water/powder stream was directed at the soft tissues or into the sulcus. Do not modify powder with additives. Additives may clog your air polishing system.

**PRECAUTIONS:** Direct contact of prophy powder with surfaces and marginal areas of dental restorations should be avoided.

**ADVERSE REACTIONS:** Allergic reactions to components of Cavitron Brand Prophylaxis Powders have been known to occur. If an allergic reaction occurs, seek medical attention immediately.

**DIRECTIONS:** Shake well before using. Use per polishing unit manufacturer's instructions.

**CONTENTS:**

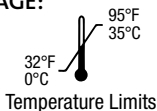
**Prophy-Jet** Prophylaxis Powder contains Sodium Bicarbonate, Flavoring, Sodium Saccharin.

**JET-Fresh** Prophylaxis Powder contains Aluminum Trihydroxide and Flavoring.

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