DELTON®

Light Curing Pit and Fissure Sealant Direct Delivery System

Caution: United States Federal law restricts this device for sale by, or on the order of, a licensed dental professional.

DELTON® Direct Delivery System provides a quick and simple application technique for DELTON® Pit and Fissure Sealants. The Direct Delivery System is available for the following formulations:

DELTON® – unfilled light curing sealant available in clear and opaque.
DELTON® PLUS – a 38% filled light curing sealant with releasable fluoride available in opaque.

COMPOSITION:

DELTON®

Aromatic and aliphatic dimethacrylate monomers
Titanium Dioxide (opaque)
Silicon Dioxide (opaque)
Initiators
Stabilizers

DELTON® PLUS

Low viscosity monomers
Triethylene glycol dimethacrylate
BisGMA
Barium alumino fluoroaluminosilicate glass
Titanium Dioxide (Plus)
Silicon Dioxide (Plus)
Sodium Fluoride (Plus)
Initiators
Stabilizers

INDICATIONS FOR USE:

Preventive sealing of pits and fissures in the primary and secondary dentition in combination with the acid-etch technique.

CONTRAINDICATIONS:

Known allergy to any components
Not to be used in case of caries

WARNINGS:

DELTON® Pit and Fissure Sealants contain methacrylates which may be irritating to the skin and eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention. After contact with skin, wash immediately with plenty of soap and water. The product may cause sensitization by skin contact in susceptible persons. If skin sensitization occurs discontinue use.

DELTON® Etchant Solution and Gel formulations contain phosphoric acid which may cause burns. Avoid contact with oral tissues, eyes and skin. If accidental contact occurs, flush affected area with generous amounts of water. In case of contact with eyes, immediately rinse with plenty of water and seek medical attention.

PRECAUTIONS:

DELTON® Etchant Solution and Gel formulations should be tightly closed immediately after use. Keep all DELTON® Pit and Fissure Sealants and DELTON® Etchant Solution and Gel out of direct sunlight. Recommended storage is 5° C – 24° C (40° F – 75° F). Recommended usage is Room Temperature.

ADVERSE REACTIONS:

Since reversible inflammatory changes of the oral mucosa may occur after accidental contact with acrylate monomers, avoid contact of DELTON® Pit and Fissure Sealants with mucous membranes.

Anaphylactic reactions may occur in susceptible individuals. If patient shows signs of anaphylaxis, seek medical attention immediately.

Step-by-Step Instructions

Cleaning

Clean surfaces to be sealed with an air-prophylaxis device (Cavitrone® Prophyl-Jet or Cavitrone® JET Plus) or an oil-free paste (NUPRO® Prophyl Paste) and a disposable prophyl angle (NUPRO® revolva®) (Fig. 1). Rinse well with water.

Isolating

Isolate the teeth to be sealed with a rubber dam or cotton rolls and dry each tooth with air, free of oil or water contamination (Fig. 2).

Etching

Apply etchant into the pits and fissures with a disposable brush or brush tip applicator (Fig. 3). Etch each tooth for 30 to 60 seconds. Rinse teeth thoroughly for at least 30 seconds each. If cotton rolls are used, they should be changed without contaminating the treated surfaces. Dry conditioned areas with air, free of oil or water contamination. Properly etched enamel has a dull frosty-white appearance. If this is not the case, re-etch enamel for another 20 seconds. Once the surfaces have been properly treated, they must be kept uncontaminated. If salivary contamination occurs, thoroughly clean with forceful water spray and dry.

Application of DELTON® Pit and Fissure Sealants

Insert the disposable Direct Delivery System (DDS) with or without Brush Tip into the DELTON® Applicator Handle. The tip of the DDS can be bent to allow easy access (Fig. 4). Remove cap. Touch the DDS tip to the surface to be sealed. Depress lever slowly while moving the DDS tip along the fissures to dispense the sealant (Fig. 5).

Curing

Cure each surface for at least 20 seconds with a polymerization unit keeping the light exit window as near as possible to the tooth without touching it (Fig. 6).

Retention Control

Check sealed area with an explorer for complete coverage and retention.

DELTON® Applicator Handle Care

Cleaning

- Carefully remove the spent DDS tip from handle and discard.
- Place handle in an ultrasonic cleaner basket and immerse in a multi-purpose cleaner.
- Operate the ultrasonic cleaner in accordance with the manufacturer's directions.
- Remove from cleaner and dry completely.

Autoclaving

- Cycle in a steam autoclave at a temperature and pressure of 273°F/31 psi (134°C/216kPa) for 15-20 minutes followed by a drying time of 20-30 minutes.

BATCH NUMBER AND EXPIRY DATE

Do not use after expiry date.

The batch number indicates important production information. The number should be quoted in all correspondence which requires identification of the product.

Manufactured By
DENTSPLY Professional
DENTSPLY International
1301 Smile Way
York, PA 17404
Tel: 1-800-989-8826

Imported and Distributed By
DENTSPLY Canada
Woodbridge, Ontario
L4L 4A3

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DENTSPLY DeTrey GmbH