DELTON®

Self Curing Pit and Fissure Sealant Lift and Place Delivery System

**Type I – Chemically Cured
Available in Opaque Formulation.
Shake bottle well before using.**

Rx Only. For Dental Use Only.

DELTON® Self Curing Pit & Fissure Sealant is a thin plastic coating which is applied to the occlusal tooth surface.

COMPOSITION:

Catalyst Liquid

Aromatic and aliphatic dimethacrylate monomers
2,6-di-tert-butyl-p-cresol (BHT)
Benzoyl Peroxide

Universal Liquid

Aromatic and aliphatic dimethacrylate monomers
2,6-di-tert-butyl-p-cresol (BHT)
n-hydroxyethyl-3-4-dimethylamline
Silicon Dioxide (opaque sealant)
Titanium Dioxide (opaque sealant)
Annatto (tinted sealant)

INDICATIONS FOR USE:

Preventive sealing of pits and fissures in the primary and secondary dentition in combination with the acid-etch technique.

CONTRAINDICATIONS:

Known allergy to any components
Not to be used in case of caries

WARNINGS:

DELTON® Sealant contains methacrylates which may be irritating to the skin and eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention. After contact with skin, wash immediately with plenty of soap and water. The product may cause sensitization by skin contact in susceptible persons. If skin sensitization occurs, discontinue use.

DELTON® Sealant Etchant contains phosphoric acid which may cause burns. Avoid contact with oral tissues, eyes and skin. If accidental contact occurs, flush affected area with generous amounts of water. In case of contact with eyes, immediately rinse with plenty of water and seek medical attention.

PRECAUTIONS:

DELTON® Sealant and DELTON® Sealant Etchant bottles should be tightly closed immediately after use. Keep out of direct sunlight. Recommended storage is 5°C-24°C (40°F-75°F). Use at room temperature.

ADVERSE REACTIONS:

Since reversible inflammatory changes of the oral mucosa may occur after accidental contact with acrylate monomers, avoid contact of DELTON® with mucous membranes.

Anaphylactic reactions may occur in susceptible individuals. If patient shows signs of anaphylaxis, seek medical attention immediately.

Directions for Use

Step-by-Step Instructions

Cleaning

Clean surfaces to be sealed with an air-prophylaxis device (Cavitron® Prophyl JET or Cavitron® JET Plus) or an oil-free paste (NUPRO® Prophyl Paste) and a disposable prophyl angle (NUPRO® revolva®) (Fig. 1). Rinse well with water.

Isolating

Isolate the teeth to be sealed with rubber dam or cotton rolls and dry each tooth with air, free of oil or water contamination (Fig. 2).

Etching

Apply etchant into the pits and fissures with a brush (Fig. 3). Etch each tooth for 30 to 60 seconds. Rinse teeth thoroughly for at least 30 seconds each. If cotton rolls are used, they should be changed without contaminating the treated surfaces. Dry conditioned areas with air, free of oil or water contamination. Properly etched enamel has a dull, frosty/white appearance. If this is not the case, re-etch enamel for another 20 seconds. Once the surfaces have been properly treated, they must be kept uncontaminated. If salivary contamination occurs, thoroughly clean with forceful water spray and dry.

Application of DELTON® Sealant

Hold bottle vertically and shake well. Dispense one drop of Universal Liquid and one drop of Catalyst Liquid into mixing well (Fig. 4). Mix thoroughly for 10 to 15 seconds.

Working Time: 45 seconds. Insert a disposable tube into the end of the applicator handle and depress lever (Fig. 5). Immerse the tip of the tube in the mixture of Universal and Catalyst and release lever. Applicator will draw up a measured amount suitable for an occlusal surface. Touch the applicator tip to the surface to be sealed. Depress lever gradually while moving the tube slowly along fissures (Fig. 6).

Curing

Set Time: 60 - 90 seconds. Check for setting of unused sealant in the mixing well. Set sealant appears as a polymerized thin film.

Retention control

Check with an explorer for complete coverage and retention. Discard applicator tube. Cold sterilize/disinfect applicator handle following solution manufacturer's recommendations. Do not autoclave.

BATCH NUMBER AND EXPIRY DATE

Do not use after expiry date.
The batch number indicates important production information. The number should be quoted in all correspondence which requires identification of the product.

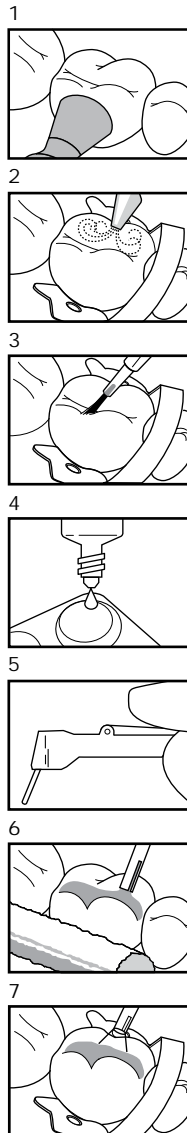
Manufactured by
DENTSPLY Professional
DENTSPLY International
York, PA 17404
Tel: 1-800-989-8826

Distributed by
DENTSPLY Canada
Woodbridge, Ontario
L4L 4A3

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