# Rinn® XCP-ORA® One Ring & Arm Positioning System

For XCP®, XCP-DS®, XCP-QT® and XCP-DS FIT® Biteblocks

550771 1 Set  
550772 2 Sets  
550777 6 Sets

Positioning Ring and Arm Kit  
Positionierungs- Ringsatz- und -Arm  
Positionnement du kit de l'anneau et de bras  
Kit anello di posizionamento con braccio  
Juego de anillo posicionador y brazo

REF	Description
550771	XCP-ORA® Arm & Ring, 1-pk
550772	XCP-ORA® Arm & Ring, 2-pk
550777	XCP-ORA® Arm & Ring, 6-pk
550773	XCP-ORA® Ring only, 1-pk

REF	Description
550774	XCP-ORA® Arm only, 1-pk
550598	Endo Ring, 1-pk
550597	Endo Arm, 1-pk
550500	Rinn® Universal Digital Sensor Covers, 500-pk

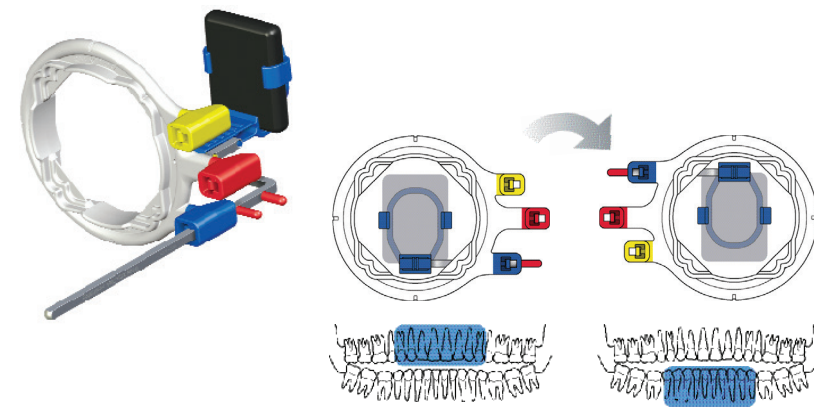
Dentsply RINN  
1301 Smile Way  
York, PA 17404, USA

**EC REP** DENTSPLY Limited  
Building 3, The Heights,  
Weybridge, Surrey KT 13 ONY  
United Kingdom

Rinn®, XCP®, XCP-DS FIT®, XCP-QT® and XCP-ORA® are trademarks of DENTSPLY International Inc. ©2017 Dentsply Sirona Preventive Printed in the USA 055907 Rev. 6 (0117)

Vertical Anterior    Vertikal Frontzahn    Vertical Antérieur    Verticale anteriore    Vertical anterior


NOTE: Depending on palate, biteblock may flex for improved patient comfort. When this occurs, adjust vertical angulation of the tube head 5-10°.

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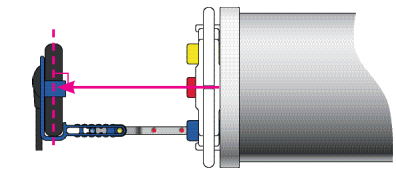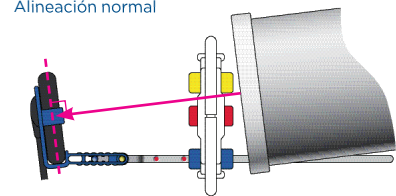
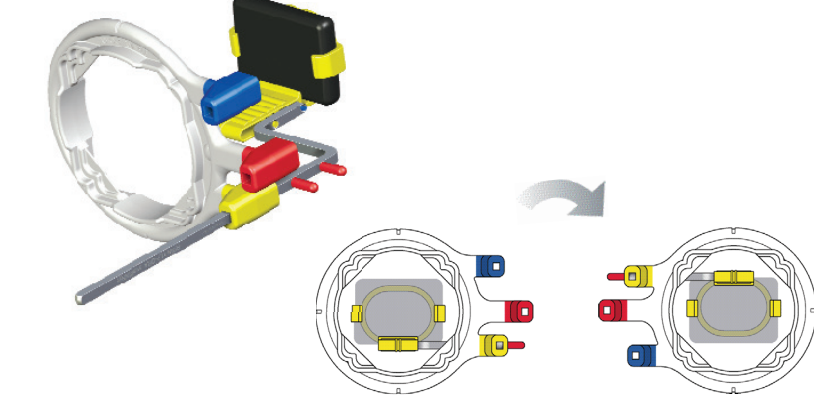
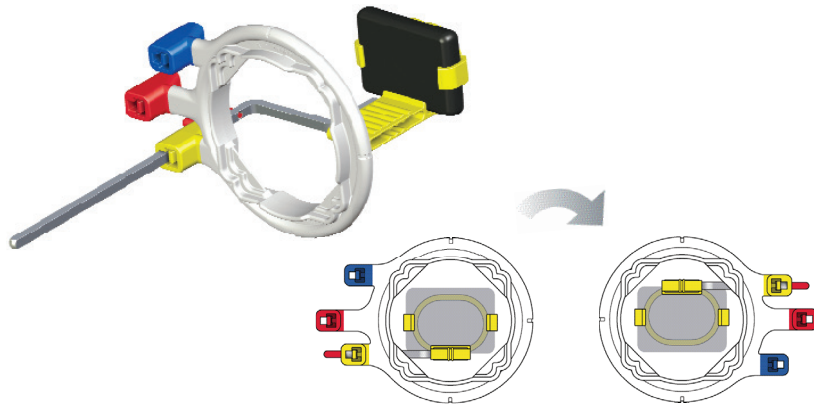
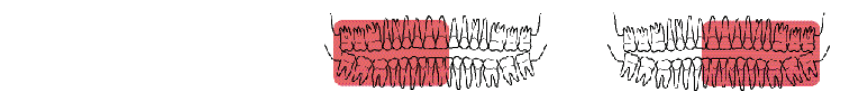
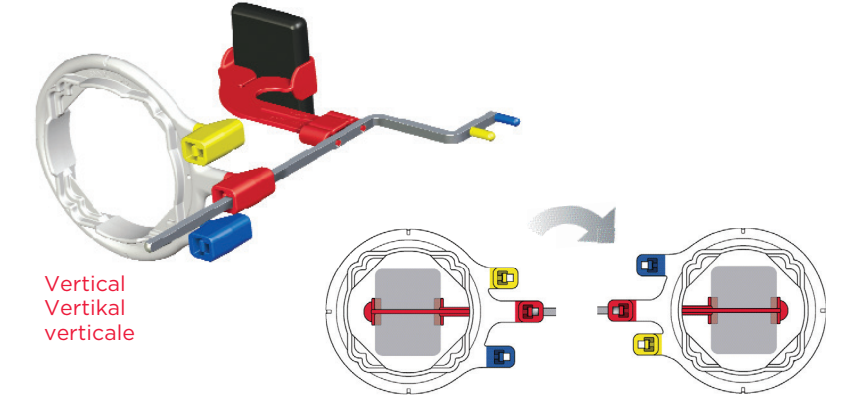
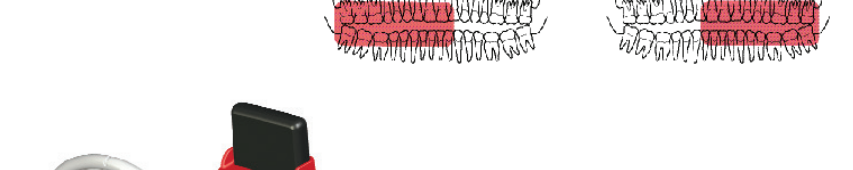
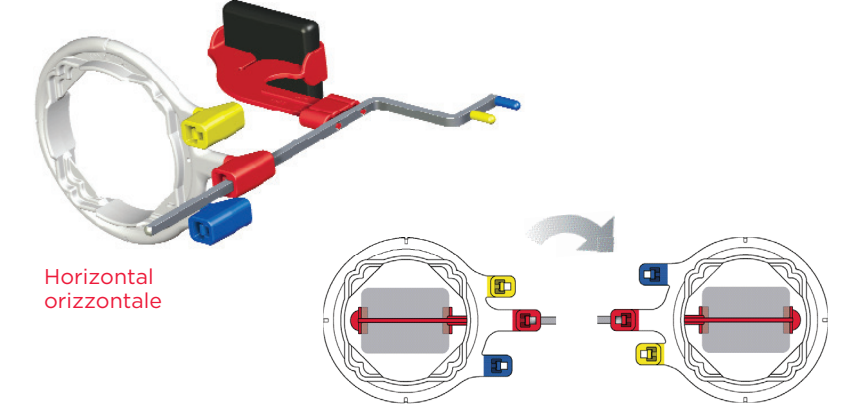
Bitewing    Bißflügel    Bitewing    Aletta    Aleta de mordida

**Caution:** U.S. Federal laws restrict this device to sale to or on the order of a licensed dental professional.

**Indications for use:** XCP® and XCP-ORA® arms and rings are indicated for positioning and aligning dental intraoral radiographs.

**Contraindications:** None known.

**Warnings:** Sterilize before use. It is the user's responsibility to qualify alternative methods of sterilization or disinfection. Failure to follow properly validated sterilization processes and approved aseptic techniques may result in cross contamination.

**Precautions:** Follow sterilization instructions carefully to prevent plastic damage during sterilization.

**Adverse Reactions:** None known.

#### Step-by-Step Instructions:

1. Insert positioning arm into the appropriate color-coded channel of the aiming ring.
2. Attach appropriate biteblock for area to be imaged. Correctly assembled, the imaging area should be centered when viewed through the aiming ring.
3. Position holder in patient's mouth and slide ring close to patient's face.
4. Align x-ray tube to ring.
5. Take exposure.
6. Adjust as needed for the next desired position.

**Reprocessing:** Disassemble instrument(s) and separate all parts prior to reprocessing.

#### Manual Cleaning and Disinfection:

1. Rinse components under running tap water using a soft-bristled brush to remove gross soil using an enzymatic detergent per manufacturer's recommendation, paying special attention to any cavities or holes.
2. Rinse under running tap water and dry with cloth or disposable wipe.
3. After thoroughly cleaning, wipe all surfaces with a disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution (such as VoloWipe) following manufacturer's instructions. Pay special attention to any cavities or holes.
4. Rinse components by wiping them with a cloth soaked in pure water or by running under running tap water.

#### Automated Cleaning and Disinfection:

As an alternative to manual cleaning and disinfection, the devices can also be processed in a washer-disinfector. Use only properly maintained, calibrated, and approved washer-disin-

factors according to ISO 15883-1.

1. Place devices in washer-disinfector allowing water and detergent to wet all surfaces and to drain off afterwards during washer-disinfector program following manufacturer's instructions. Run a washer disinfector program with AO value  $\geq$  600.

#### Sterilization:

1. Put components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal arms to avoid melting or warping.
2. Cycle steam autoclave at 270°F (132°C) for 10 minutes at 30 PSI (206.8 kPa). Do not exceed 273°F (134°C).
3. To maintain sterility, instruments should remain bagged until ready for use.

**Storage:** Store at room temperature (18-25°C / 65-77°F).

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