Seal&Protect®

Protective Sealant for Exposed Dentin

Versiegelungsschutz für freiliegendes Dentin
Matériau protecteur pour la dentine exposée
Sigillante protettivo per dentina esposta
Sellador protector para dentina expuesta
Защитное покрытие для чувствительного дентина
Selante de protecção para dentina exposta
Skyddslack för exponerat dentin
Beschermdende Sealant voor blootliggend dentine
Beskyttende Sealer til eksponeret dentin
Açığa çıkmış dentin için koruyucu sealant
Beskyttende forsegling for eksponert dentin
Apsauginis atviro dentino silantas

Instructions for Use_____	English	2
Gebrauchsanweisung _____	Deutsch	7
Mode d'emploi _____	Français	12
Istruzioni per l'uso_____	Italiano	17
Instrucciones de uso_____	Español	22
Инструкция по применению _____	Русский	27
Instruções de Utilização _____	Português	32
Bruksanvisning _____	Svenska	37
Gebruiksaanwijzing_____	Nederlands	42
Brugsanvisning _____	Dansk	47
Kullanım Talimatları_____	Türkçe	52
Bruksanvisning _____	Norsk	57
Naudojimo instrukcijos _____	Lietuvių k.	62

Seal&Protect®

Protective Sealant for Exposed Dentin

CAUTION: For dental use only.

USA: Rx only

Content.....	Page
1 Product description.....	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene.....	6
5 Lot number and expiration date	6

1 Product description

Seal&Protect® is a protective sealant for exposed root dentin.

Seal&Protect® reduces:

- Wear of cervical dentin by infiltrating and coating the dentin thus increasing its mechanical strength.
- Cervical hypersensitivity by blocking dentinal tubules.
- Dentin decalcification by acids. Seal&Protect® has an antimicrobial effect against caries-associated bacteria. This reduces plaque accumulation on the sealed surface.

1.1 Delivery forms

- Easy squeeze bottle

1.2 Composition

- Urethane dimethacrylate (UDMA)
- Trimethacrylate
- Phosphoric acid modified acrylate resin (PENTA)
- Highly dispersed silicon dioxide
- Camphorquinone (photoinitiator)
- Ethyl-4(dimethylamino)benzoate (photoaccelerator)
- Butylated hydroxy toluene (BHT)
- Cetylamine hydrofluoride
- Triclosan
- Acetone

1.3 Indications

- Reduction of abrasion, erosion and abfraction of exposed cervical dentin.
- Treatment of hypersensitive cervical areas.

1.4 Contraindications

Use with patients who have a known hypersensitivity to any of the components.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains polymerizable acrylates and methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

The material contains acetone which may be irritating to eyes and is highly flammable.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitisation or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.
- **Do not breathe in vapor.**

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close bottle immediately after use.
- Use in a well-ventilated area. Avoid inhaling vapor.
- Highly flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Avoid the product saturating the gingival retraction cord. If it soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
- Applicator tips and Dentsply Sirona applicator dish are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

- Product is a light cured material. Protect from ambient light.
- Interactions:
 - Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.
 - If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of direct sunlight and store in a well-ventilated place.
- Store at temperatures between 10 °C and 24 °C (50 °F and 75 °F). Use the product at room temperature.
- Do not freeze.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Cleaning

1. Clean uninstrumented enamel and dentin with a rubber cup and pumice or a non-fluoride cleaning paste such as Nupro[®] prophylaxis paste.
2. Remove prophylaxis paste with air/water spray.
3. Isolate area to be treated with cotton rolls. In the case of mandibular teeth, place saliva ejector.
4. Clean freshly instrumented enamel and dentin with water spray, then air dry.
5. Dry cleaned area with a 2-second blow of air free of oil or water contamination.
6. Do not desiccate the tooth substance – cavity surface should be moist.

3.2 Application and curing

Cross-contamination.

Infection.

1. To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled outside the dental unit with clean/disinfected gloves.

1. Dispense Seal&Protect[®] onto a fresh Dentsply Sirona applicator tip or into a fresh Dentsply Sirona applicator dish¹ or standard dappen dish. Per surface to be treated, two to three drops are required. It is recommended to treat no more than three adjacent surfaces per application procedure.
2. Immediately apply ample amounts of Seal&Protect[®] to the dentin surface with a Dentsply Sirona applicator tip¹. Thoroughly saturate the exposed dentin.
3. Leave the surface undisturbed for 20 seconds.
4. Remove solvent by blowing gently with air from a dental syringe for at least 5 seconds.
5. Cure Seal&Protect[®] for 10 seconds².

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

6. Apply a second layer of Seal&Protect[®].
7. Remove solvent from second layer by blowing gently with air from a dental syringe for at least 5 seconds.
8. Cure Seal&Protect[®] for 10 seconds².
9. Remove oxygen-inhibited (soft surface) layer with a cotton pellet or cotton roll.
10. Check for excess material in the gingival sulcus. Remove excess material with a periodontal probe, scaler or curet.
11. Discharge used Dentsply Sirona applicator tip and Dentsply Sirona applicator dish.

¹ Dentsply Sirona applicator dish and applicator tips are available from your dental dealer.

² Check curing light for minimum light output of at least 800 mW/cm². Cure for at least 20 seconds if light output is between 500 and 800 mW/cm².

It is recommended that all treated surfaces be checked for partial or total loss of the sealant during a second appointment. Depending on individual oral hygiene, wear of the sealant will require re-application every three to six months.

4 Hygiene

Cross-contamination.

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.

5 Lot number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: "YYYY-MM"
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

© Dentsply Sirona 2014-01-31

[These Instructions for Use are based on Master Version 02]

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Dentsply DeTrey GmbH